


Correcte bloedafname Streck tubes:

Voor de CLARIGO test is het noodzakelijk bloed te collecteren in Streck-tubes:

*Graag erop letten dat de STRECK tubes voor de NIPT zo goed als volledig gevuld zijn, zodat er voldoende plasma kan geïsoleerd worden. Er staat geen merkstreep op de tubes maar zo lang mogelijk laten vullen (vullen langzaam) en daarna voorzichtig mengen zodat de vloeistof (reeds aanwezig in de tubes) voldoende gemengd wordt met het bloed zoals hieronder aangeduid op te tekening (dus niet kort mengen zoals men met EDTA doet maar langzaam zodat er geen hemolyse ontstaat!).*

3. Fill tube completely.
4. Remove tube from adapter and immediately mix by gentle inversion 8 to 10 times. Inadequate or delayed mixing may result in incorrect analytical results or poor product performance. One inversion is a complete turn of the wrist, 180 degrees, and back per the figure below:

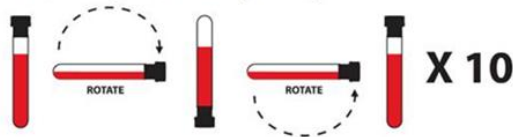

5. After collection, transport and store tubes within the recommended temperature range.

Na staalafname dient plasma geïsoleerd te worden binnen de 48h (gebeurt in het moleculair labo).